DIESSE Diagnostica Senese SpA

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Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

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(EN)

The present instructions refer to the control sera listed below:

Product	REF
CHORUS CHLAMYDIA CONTROL SERUM <i>Product included in the Annex. IIB, Dir. 98/79 CE</i> Notified Body Code Number 0123	85832 0123
CHORUS ADENOVIRUS CONTROL SERUM	85830
CHORUS BORDETELLA PERTUSSIS CONTROL SERUM	85894
CHORUS BORRELLIA CONTROL SERUM	85902
CHORUS CAMPYLOBACTER CONTROL SERUM	85895
CHORUS COXSACKIE B CONTROL SERUM	85903
CHORUS INFLUENZA A CONTROL SERUM	85872
CHORUS INFLUENZA B CONTROL SERUM	85873
CHORUS LEGIONELLA PNEUMOPHYLA CONTROL SERUM	85874
CHORUS LEPTOSPIRA CONTROL SERUM	85896
CHORUS LISTERIA MONOCYTOGENES CONTROL SERUM	85875
CHORUS MYCOPLASMA PNEUMONIAE CONTROL SERUM	85877
CHORUS PARAINFLUENZA CONTROL SERUM	85893
CHORUS Q-FEVER CONTROL SERUM	85886
CHORUS RESPIRATORY SYNCYTIAL CONTROL SERUM	85889
CHORUS SHIGELLA CONTROL SERUM	85932
CHORUS YERSINIA CONTROL SERUM	85931

COMPOSITION
CONTROL + CFT-MAT POSITIVE CONTROL

1 vial x 0,5 ml

Contents: Positive control serum at known titer (the titer is reported on the label).**Preparation:** ready for use. To be used as any other sample.
STORAGE AND STABILITY OF REAGENT

Control sera must be stored at 2 - 8 °C.

The expiry date is printed on the label. After opening, the reagent is stable for 2 months at 2 - 8 °C.

After opening the control sera can be stored at -20°C in small aliquots, to avoid repeated thawing and freezing. In this case they are stable for 3 years.

PRECAUTIONS

ONLY FOR IN VITRO DIAGNOSTIC USE.

All the sera are of animal origin and have been tested for the absence of possible infecting agents. In any case, all sera must be treated as potentially infectious.

PROCEDURE

Decomplement for 30 minutes at 56 °C and use as any other sample being tested.

Attention: In semiquantitative serological assays, a result +/- 1 dilution with respect to the declared value is generally accepted.

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