**DIESE Diagnostica Senese SpA**

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Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

**CHORUS CFT CONTROL SERUM**

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**DIESSE Diagnostica Senese SpA**

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Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

**CHORUS CFT CONTROL SERUM**
**Instructions for use of the Control Sera for the complement fixation test (CFT)**

(EN)

The present instructions refer to the control sera listed below:

Product	REF
CHORUS CHLAMYDIA CONTROL SERUM	85832
<i>Product included in the Annex. IIB , Dir. 98/79 CE</i>	
<i>Notified Body Code Number 0123</i>	
CHORUS ADENOVIRUS CONTROL SERUM	85830
CHORUS BORDETELLA PERTUSSIS CONTROL SERUM	85894
CHORUS BORRELLIA CONTROL SERUM	85902
CHORUS CAMPYLOBACTER CONTROL SERUM	85895
CHORUS COXSACKIE B CONTROL SERUM	85903
CHORUS INFLUENZA A CONTROL SERUM	85872
CHORUS INFLUENZA B CONTROL SERUM	85873
CHORUS LEGIONELLA PNEUMOPHYLA CONTROL SERUM	85874
CHORUS LEPTOSPIRA CONTROL SERUM	85896
CHORUS LISTERIA MONOCYTOGENES CONTROL SERUM	85875
CHORUS MYCOPLASMA PNEUMONIAE CONTROL SERUM	85877
CHORUS PARAINFLUENZA CONTROL SERUM	85893
CHORUS Q-FEVER CONTROL SERUM	85886
CHORUS RESPIRATORY SYNCYTIAL CONTROL SERUM	85889
CHORUS SHIGELLA CONTROL SERUM	85932
CHORUS YERSINIA CONTROL SERUM	85931

**COMPOSITION**
**CONTROL + CFT-MAT POSITIVE CONTROL**

1 vial x 0,5 ml

**Contents:** Positive control serum at known titer (the titer is reported on the label).

Preparation: ready for use. To be used as any other sample.

**STORAGE AND STABILITY OF REAGENT**
**Control sera must be stored at 2 - 8 °C.**

The expiry date is printed on the label. After opening, the reagent is stable for 2 months at 2 - 8 °C.

After opening the control sera can be stored at -20°C in small aliquots, to avoid repeated thawing and freezing. In this case they are stable for 3 years.

**PRECAUTIONS**

ONLY FOR IN VITRO DIAGNOSTIC USE.

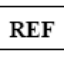
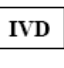
All the sera are of animal origin and have been tested for the absence of possible infecting agents.

In any case, all sera must be treated as potentially infectious.

**PROCEDURE**

Decomplement for 30 minutes at 56 °C and use as any other sample being tested.

**Attention:** In semiquantitative serological assays, a result +/- 1 dilution with respect to the declared value is generally accepted.

	EN ES IT	Date of manufacture Fecha de fabricación Data di fabbricazione	FR GR PT	Date de fabrication Ημερομηνία Παραγωγής Data de fabrico
	EN ES IT	Use By Fecha de caducidad Utilizzare entro	FR GR PT	Utiliser jusque Ημερομηνία λήξης Prazo de validade
	EN ES IT	Do not reuse No reutilizar Non riutilizzare	FR GR PT	Ne pas réutiliser Μην κάνετε επαναληπτική χρήση Não reutilizar
	EN ES IT	Caution, consult accompanying documents Atención, ver instrucciones de uso Attenzione, vedere le istruzioni per l'uso	FR GR PT	Attention voir notice d'instructions Προειδοποίηση, συμβουλευτείτε τα συνοδά έντυπα Atenção, consulte a documentação incluída
	EN ES IT	Manufacturer Fabricante Fabbicante	FR GR PT	Fabricant Κατασκευαστής Fabricante
	EN ES IT	Contains sufficient for <n> tests Contenido suficiente para <n> ensayos Contenuto sufficiente per "n" saggi	FR GR PT	Contenu suffisant pour "n" tests Περιεχόμενο επαρκές για «n» εξετάσεις Conteúdo suficiente para "n" ensaios
	EN ES IT	Temperature limitation Límite de temperatura Limiti di temperatura	FR GR PT	Limites de température Περιορισμοί θερμοκρασίας Limites de temperatura
	EN ES IT	Consult Instructions for Use Consulte las instrucciones de uso Consultare le istruzioni per l'uso	FR GR PT	Consulter les instructions d'utilisation Συμβουλευτείτε τις οδηγίες χρήσης Consulte as instruções de utilização
	EN ES IT	Biological risks Riesgo biológico Rischio biologico	FR GR PT	Risques biologiques Βιολογικοί κίνδυνοι Risco biológico
	EN ES IT	Catalogue number Número de catálogo Numero di catalogo	FR GR PT	Référence du catalogue Αριθμός καταλόγου Referência de catálogo
	EN ES IT	In Vitro Diagnostic Medical Device Producto sanitario para diagnóstico in vitro Dispositivo medico-diagnostico in vitro	FR GR PT	Dispositif médical de diagnostic in vitro In Vitro Διαγνωστικό Ιατροτεχνολογικό προϊόν Dispositivo médico para diagnóstico in vitro
	EN ES IT	Devices Dispositivos Dispositivi	FR GR PT	Dispositifs Συσκευές Dispositivos
	EN ES IT	Batch code Código de lote Codice del lotto	FR GR PT	Code du lot Αριθμός Παρτίδας Código do lote

**DIESSE Diagnostica Senese**  
**Via delle Rose 10**  
**53035 Monteriggioni (Siena) Italy**  
**Tel. 0577-587111**