CHORUS SANITIZING SOLUTION

REF 83604

DIESSE

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INSTRUCTIONS FOR USE

CHORUS SANITIZING SOLUTION

Accessory reagent for use in CHORUS instruments

For *In Vitro* Diagnostic Use Only

1. INTENDED PURPOSE

To be used in CHORUS instruments as a tube sanitizing solution.

It must only be used by professional laboratory personnel.

SANIT SOL 1000x SANITIZING SOLUTION 1000X 1x20 mL

INTERCHANGEABLE BETWEEN BATCHES

Contents: Complex emulsion of anionic and non-ionic surfactant agents, stabilizing agents, alkalis, non-phosphate emulsifying agents and first quality sequestering agents in watery base. It contains <3% Potassium Hydroxide (KOH).

Preparation: Dilute the required volume 1:1000 with distilled water to obtain the Sanitizing Solution ready for use.

It is necessary to prepare 150 ml of ready-to-use Sanitizing Solution for each cycle of sanitization.

2. USE

1. In the **CHECK** menu, select the **SANIT** key

2. Follow the instructions on the monitor

Estimated time: 2 hours

Conduct this operation once a week.

3. STABILITY AFTER OPENING

Concentrated solution: stable up to the expiry date printed on the label.

Ready-to-use solution: stable up to 24 hours at the temperature of 15-30 °C.

4. WARNINGS

Withdraw taking care not to contaminate the product.

Regarding the safety characteristics, consult the Safety Data Sheet (www.diesse.it).

5. INCIDENT REPORTING

If a serious incident has occurred in relation to this device in the European Union market territory, please report it to the manufacturer and the competent authority of its Member State without delay.

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