DIESSE Diagnostica Senese SpA

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Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

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DIESSE Diagnostica Senese SpA

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Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

INSTRUCTIONS FOR USE**STREPTOSLIDE LATEX**

(English)

REF 0211	STREPTOSLIDE A	REF 0214	STREPTOSLIDE D
REF 0212	STREPTOSLIDE B	REF 0215	STREPTOSLIDE F
REF 0213	STREPTOSLIDE C	REF 0216	STREPTOSLIDE G

The products reported above are supplementary reagents for use with Streptoslide Extracting Reagents **REF** 0217.

1. INTENDED USE

SLIDE TEST FOR THE IDENTIFICATION OF STREPTOCOCCAL GROUPS A, B, C, D, F AND G.

2. INTRODUCTION

Lancefield demonstrated that streptococci have group specific antigens on their cellular wall. These antigens (polysaccharides) have been extracted and recognised by specific homologous antisera. Most of the human pathogenic streptococci belong to the groups A,B,C,D,F and G (1). Group E streptococci are rarely isolated. The extraction procedures have been improved and several reagents based on chemical as well as enzymatic reactions are now available (1,2,3,4).

In the STREPTOSLIDE kit, the identification of Streptococci group A, B, C, F and G is performed without incubation by a simple chemical extraction (micromethod with nitrous acid) using latex with group specific antibodies. Group D is usually identified by direct reaction with the specific latex reagent for Group D. The isolated colonies negative to all group specific reagents and suspected to belong to the Group D, are tested again with the specific Group D latex reagent after enzymatic digestion.

3. PRINCIPLE OF THE METHOD

A few isolated colonies of streptococci are suspended in the reaction mixture for extraction of the group antigen. An aliquot of the suspension is mixed with latex reagent coated with immunoglobulins specific for the group antigens. Agglutination allows identification of the group.

4. KIT CONTENTS

Each product is sufficient for 50 tests.

Bring to room temperature before use.

LATEX GROUPING LATEX A, B, C, D, F, or G 1 dropper dispenser x 2.5 mL

Contents: Suspensions of polystyrene latex particles coated with specific immunoglobulins (rabbit) for group antigens A,B,C,D,F or G, respectively. Ready for use.

FOR USE, SEE THE INSTRUCTIONS FOR USE OF THE KIT **REF 0217 STREPTOSLIDE EXTRACTING REAGENTS.**

DIESSE Diagnostica Senese SpA
Via delle Rose 10, 53035 Monteriggioni SI
Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

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