DIESSE Diagnostica Senese SpA

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Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

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Serological studies of patients with persistent fever can be useful in detecting typhoid fever and brucellosis (1,2).

The test consists in the observation of a direct agglutination reaction between the bacterial suspension and the serum sample which is suspected of containing specific antibodies. The present method, known as the Widal-Wright reaction, allows a semiquantitative titration of the serum agglutinins.

3. PRINCIPLE OF THE METHOD

The serum sample is mixed with a bacterial suspension at the optimal concentration for the slide test. After 1 minute a visible agglutination will appear if the serum contains specific antibodies against the bacteria in the suspension. Positive samples must then be tested in the tube test, using one drop of the same suspension.

4. KIT COMPOSITION

- The Positive Control is sufficient for 50 tests.
- Bring the reagents to room temperature before use.

CONTROL + W.W. Positive Control 1 x 1 mL

Contents: Polyvalent control serum of animal origin, containing sodium azide 0,09% as preservative. The serum contains antibodies which agglutinate the following bacterial suspensions: *S. typhi*, *S. paratyphi A*, *S. paratyphi B* and *Brucella*. Ready for use.

Use: The agglutinating serum is used as a positive control serum for the coloured bacterial suspensions to be used in the Widal Wright test.

5. ANALYTICAL WARNINGS

In vitro diagnostic reagent.

- Bring to room temperature before use.
- Take care to avoid bacterial or chemical contamination of the reagent.
- Discard the product if it appears very turbid or maleodorous.

6. STORAGE AND STABILITY AFTER OPENING

Store at 2/8° C.

Stable after opening up to the expiry date if stored correctly.

7. PROCEDURE

Dispense 20 µL of control serum on a slide; add one drop of reagent and mix, turning the slide around for 1 minute. Evident agglutination will be seen.

8. QUALITY CONTROL

If the serum does not cause agglutination of the bacterial suspension used in the WW, there is probably a defect of the bacterial suspensions.

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