DIESE Diagnostica Senese SpA
Via delle Rose 10, 53035 Monteriggioni SI
Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

CHORUS CONTROL SERA

Produits inclus dans l'annexe IIB , Dir. 98/79 CE	REF
Προϊόντα που περιλαμβάνονται στο Παρ. IIB, Οδηγ. 98/79/EK	
CHORUS CYTOMEGALOVIRUS IgG CONTROL SERUM	81500
CHORUS CYTOMEGALOVIRUS IgG AVIDITY CONTROL SERUM	81501
CHORUS CYTOMEGALOVIRUS IgM CONTROL SERUM	81502
CHORUS RUBELLA IgG CONTROL SERUM	81523
CHORUS RUBELLA IgG AVIDITY CONTROL SERUM	81524
CHORUS RUBELLA IgM CONTROL SERUM	81525
CHORUS TOXOPLASMA IgG CONTROL SERUM	81531
CHORUS TOXOPLASMA IgG AVIDITY CONTROL SERUM	81532
CHORUS TOXOPLASMA IgM CONTROL SERUM	81533
CHORUS TOXOPLASMA IgA CONTROL SERUM	81534
CHORUS CHLAMYDIA TRACHOMATIS IgG CONTROL SERUM	81542
CHORUS CHLAMYDIA TRACHOMATIS IgA CONTROL SERUM	81543
Notified Body Code Number 0123	 0123
Produit	REF
Προϊόν	
CHORUS EPSTEIN-BARR VCA IgG CONTROL SERUM	81503
CHORUS EPSTEIN-BARR VCA IgM CONTROL SERUM	81504
CHORUS EPSTEIN-BARR EBNA IgG CONTROL SERUM	81505
CHORUS EPSTEIN-BARR EARLY ANT. IgG CONTROL SERUM	81506

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Instructions for use of the Control Sera

(EN)

1. INTENDED USE

Control material to be used exclusively in the control of precision of the Chorus/Chorus TRIO instruments.

2. COMPOSITION

CONTROL + POSITIVE CONTROL 2 x 0.4 mL

Contents: Reactive human serum for the parameters reported above.

Preparation: ready for use. To be used as any other sample.

3. STORAGE AND STABILITY OF REAGENT

Control sera must be stored at 2-8°C.

The expiry date is printed on the label.

Once opened, if properly stored at 2-8°C, the product is stable up to the expire date.

4. PRECAUTIONS

ONLY FOR IN VITRO DIAGNOSTIC USE

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HbsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling material of human origin.

Waste disposal: serum samples once used must be treated as infectious residuals and eliminated according to law.

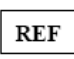
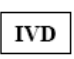
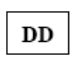
5. PROCEDURE

To be used use as any other sample being tested.

6. REFERENCES

- C24-A3 (ISBN 1-56238-613-1), Clinical and Laboratory Standard Institute (CLSI), Vol. 26 No.25, (2006)

	Section
Changes introduced in the current revision	REF

	EN ES IT	Date of manufacture Fecha de fabricación Data di fabbricazione	FR GR PT	Date de fabrication Ημερομηνία Παραγωγής Data de fabrico
	EN ES IT	Use By Fecha de caducidad Utilizzare entro	FR GR PT	Utiliser jusque Ημερομηνία λήξης Prazo de validade
	EN ES IT	Caution, consult accompanying documents Atención, ver instrucciones de uso Attenzione, vedere le istruzioni per l'uso	FR GR PT	Attention voir notice d'instructions Προειδοποίηση, συμβουλευτείτε τα συνοδά έντυπα Atenção, consulte a documentação incluída
	EN ES IT	Manufacturer Fabricante Fabbicante	FR GR PT	Fabricant Κατασκευαστής Fabricante
	EN ES IT	Contains sufficient for <n> tests Contenido suficiente para <n> ensayos Contenuto sufficiente per "n" saggi	FR GR PT	Contenu suffisant pour "n" tests Περιεχόμενο επαρκές για «n» εξετάσεις Conteúdo suficiente para "n" ensaios
	EN ES IT	Temperature limitation Límite de temperatura Limiti di temperatura	FR GR PT	Limites de température Περιορισμοί θερμοκρασίας Limites de temperatura
	EN ES IT	Consult Instructions for Use Consulte las instrucciones de uso Consultare le istruzioni per l'uso	FR GR PT	Consulter les instructions d'utilisation Συμβουλευτείτε τις οδηγίες χρήσης Consulte as instruções de utilização
	EN ES IT	Biological risks Riesgo biológico Rischio biologico	FR GR PT	Risques biologiques Βιολογικοί κίνδυνοι Risco biológico
	EN ES IT	Catalogue number Número de catálogo Numero di catalogo	FR GR PT	Référence du catalogue Αριθμός καταλόγου Referência de catálogo
	EN ES IT	In Vitro Diagnostic Medical Device Producto sanitario para diagnóstico in vitro Dispositivo medico-diagnostico in vitro	FR GR PT	Dispositif médical de diagnostic in vitro In Vitro Διαγνωστικό Ιατροτεχνολογικό προϊόν Dispositivo médico para diagnóstico in vitro
	EN ES IT	Devices Dispositivos Dispositivi	FR GR PT	Dispositifs Συσκευές Dispositivos
	EN ES IT	Batch code Código de lote Codice del lotto	FR GR PT	Code du lot Αριθμός Παρτίδας Código do lote

DIESSE Diagnostica Senese
Via delle Rose 10
53035 Monteriggioni (Siena) Italy
Tel. 0577-587111

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