CHORUS

Chikungunya IgG

DIESSE
REF 81133

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INSTRUCTIONS FOR USE

CHORUS Chikungunya IgG

For the qualitative determination of anti-Chikungunya IgG antibodies

For *In Vitro* Diagnostic Use Only

1. INTENDED USE

Immunoenzymatic method for the qualitative determination of IgG class antibodies against Chikungunya in human serum, using a disposable device applied on the Chorus and Chorus TRIO instruments.

2. INTRODUCTION

Chikungunya Virus, belonging to Togaviridae family, genus Alphavirus, causes a disease called "Chikungunya Fever" spread by bite of *Aedes albopictus* (tiger mosquito) and *Aedes aegypti* mosquito.

Chikungunya Virus often produces a mild illness that may be confused with Dengue Virus, indeed symptoms include fever, headache, arthralgia, myalgia and rash. Although serious complications are uncommon arthralgia can be debilitating and may persist for months, or even years, after infection.

The disease is endemic to Africa, India and South-East Asia, while its detection outside of these countries is normally related to trips to those areas.

The potential risks of transmission through donations of blood should also be considered. For this reason it is of great importance to conduct the serological tests for the diagnosis and for blood and organs donations.

3. PRINCIPLE OF THE METHOD

The Chorus Chikungunya IgG device is ready to use for the detection of IgG antibodies against Chikungunya, in the Chorus/Chorus TRIO instruments.

The test is based on the ELISA principle (Enzyme linked ImmunoSorbent Assay). The antigen is bound to the solid phase. The specific immunoglobulins are bound to the antigen through incubation with diluted human serum.

After washing to eliminate the proteins which have not reacted, incubation is performed with the conjugate, composed of anti-human IgG monoclonal antibodies conjugated to horseradish peroxidase. The unbound conjugate is eliminated, and the peroxidase substrate added. The blue colour which develops is proportional to the concentration of specific antibodies present in the serum sample.

The disposable devices contain all the reagents to perform the test when applied on the Chorus/Chorus TRIO instruments.

The results are expressed in Index (OD sample/OD cut-off).

4. WARNINGS AND PRECAUTIONS

FOR *IN VITRO* DIAGNOSTIC USE ONLY

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HBsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling material of human origin.

Waste disposal: serum samples, calibrators and strips once used must be treated as infectious residuals and eliminated according to law.

Health and Safety Information

1. Do not pipette by mouth.
2. Wear disposable gloves and eye protection while handling specimens.
3. Wash hands thoroughly after placing the devices in the Chorus/Chorus TRIO instrument.
4. Consult the relative Material Safety Data Sheet (available on request) for all the information on safety concerning the reagents contained in the kit.
5. Neutralized acids and other liquid waste should be decontaminated by adding a sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1%. A 30 minute exposure to 1% sodium hypochlorite may be necessary to ensure effective decontamination.
6. Spillage of potentially infectious materials should be removed immediately with adsorbent paper tissue and the contaminated area swabbed with, for example, 1% sodium hypochlorite before work is continued. Sodium hypochlorite should not be used on acid-containing spills unless the spill area is first wiped dry. Materials used to clean spills, including gloves, should be disposed of as potentially biohazardous waste. Do not autoclave materials containing sodium hypochlorite.

Analytical Precautions

Bring the devices to room temperature (18-30°C) before use; use within 60 min.

1. **Discard devices which show the substrate (well 4) blue colored.**
2. Adding the sample into the well verify that it is perfectly distributed on the bottom.
3. Check for the presence of the reagents in the device and that the device is not damaged; do not use devices which are lacking a reagent and/or present foreign bodies in the reaction well when visually inspected.
4. The devices are for use with the Chorus/Chorus TRIO instrument; the Instructions for Use must be carefully followed and the Instrument Operating Manual must be consulted.
5. Check that the Chorus/Chorus TRIO instrument is set up correctly (see Operating Manual).
6. Do not alter the bar code placed on the handle of the device in order to allow correct reading by the instrument.

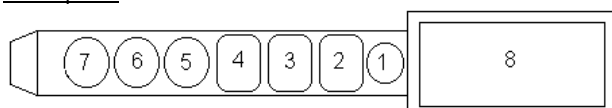
7. Avoid using self-defrosting freezers for the storage of the samples.
8. Defective barcodes can be inserted manually in the instrument (see Operating Manual).
9. Do not expose the devices to strong light or to hypochlorite vapors during storage and use.
10. The use of strongly hemolyzed, lipemic, icteric samples, of serum not completely coagulated or of samples presenting microbial contamination may all constitute sources of error.
11. Do not use the device after the expiry date.
12. **Make sure that the instrument is connected to the Washing Buffer (Ref. 83606).**

5. KIT COMPOSITION AND REAGENT PREPARATION

The kit is sufficient for 36 tests.

DD DEVICES 6 packages each containing 6 devices

Description:

Position 8: Space for application of bar code label

Position 7: Empty

Position 6: MICROPLATE WELL

Coated with Chikungunya antigen

Position 5: UNCOATED MICROPLATE WELL

Position 4: TMB SUBSTRATE

Contents: Tetramethylbenzidine 0.26 mg/ml and H₂O₂ 0.01% stabilized in 0.05 mol/L citrate buffer (pH 3.8)

Position 3: SAMPLE DILUENT

Contents: Saline proteic solution containing Proclin.

Position 2: CONJUGATE

Contents: anti-human IgG monoclonal antibodies labelled with horseradish peroxidase, in phosphate buffer containing phenol 0.05% and Bronidox 0.02%.

Position 1: EMPTY WELL

In which the operator must place the undiluted serum

Use: equilibrate a package at room temperature, open the package and remove the required devices; replace the others in the bag with the silica gel, expel the air and seal by pressing the closure. Store at 2-8°C.

CALIBRATOR CALIBRATOR 1 x 0.175 ml

Contents: Diluted human serum containing anti-Chikungunya IgG antibodies and preservative. Liquid, ready for use.

CONTROL + POSITIVE CONTROL 1 x 0.425 ml

Contents: Diluted human serum containing anti-Chikungunya IgG antibodies and preservative. Liquid, ready for use.

MATERIALS REQUIRED BUT NOT PROVIDED

- WASHING BUFFER **REF** 83606
- CLEANING SOLUTION 2000 **REF** 83609
- SANITIZING SOLUTION **REF** 83604 - 83608
- CHORUS NEGATIVE CONTROL/SAMPLE DILUENT **REF** 83607

- Chorus/Chorus TRIO Instrument
- Distilled or deionised water
- Normal laboratory glassware: cylinders, test-tubes etc.
- Micropipettes for the accurate collection of 50-200 µl solution
- Disposable gloves
- Sodium Hypochlorite solution (5%)
- Containers for collection of potentially infectious materials

6. STORAGE AND STABILITY OF REAGENTS

Reagents must be stored at 2/8°C. In the case of storage at an incorrect temperature the calibration must be repeated and the run validated using the control serum (see section 9, Test validation).

The expiry date is printed on each component and on the kit label.

Reagents have a limited stability after opening:

DEVICES	8 weeks at 2/8°C
CALIBRATOR	8 weeks at 2/8°C
POSITIVE CONTROL	8 weeks at 2/8°C

7. SPECIMEN COLLECTION AND STORAGE

The sample is composed of serum collected in the normal manner from the vein and handled with all precautions dictated by good laboratory practice.

Possible consequences, in case of use of other biological liquids, are not known.

The fresh serum may be stored for 4 days at 2/8°C, or frozen for longer periods at -20°C, and can be thawed a maximum of 3 times.

Do not keep the samples in auto-defrosting freezers. Defrosted samples must be shaken carefully before use.

Heat-inactivation can rise to erroneous results.

The quality of the sample can be seriously affected by microbial contamination which leads to erroneous results.

8. ASSAY PROCEDURE

1. Open the package (on the side containing the pressure-closure), remove the number of devices required and seal the rest in the bag after expelling the air.
2. Check the state of the device according to the indications reported in chapter 4, Analytical Precautions.
3. Dispense 50 µl of undiluted test serum in well no. 1 of each device; at each change of batch, use a device for the calibrator.
4. Place the devices in the Chorus/Chorus TRIO instrument. Perform the calibration (if necessary) and the test as reported in the Instrument Operating Manual.

9. TEST VALIDATION

Use the control serum to check the validity of the results obtained. It should be used as reported in the Instrument Operating Manual. If the instrument signals that the control serum has a value outside the acceptable range, the calibration must be repeated. The previous results will be automatically corrected.

If the result of the control serum continues to be outside the acceptable range, contact the Scientific Support.

Tel: 0039 0577 319554
 Fax: 0039 0577 366605
 email: scientificsupport@diesse.it

10. INTERPRETATION OF THE RESULTS

The Chorus/Chorus TRIO instrument expresses the result in Index (OD sample/OD cut-off).

The test on the examined serum can be interpreted as follows:

POSITIVE: when the result is > 1.2

NEGATIVE: when the result is < 0.8

DOUBTFUL/EQUIVOCAL: for all values between 0.8 and 1.2

If the result is doubtful/equivocal, repeat the test. If it remains doubtful/equivocal, collect a new serum sample.

11. LIMITATIONS

All the values obtained require a careful interpretation that must consider other indicators relative to the patient.

The test, indeed, can not be used alone for a clinical diagnosis and the test result should be evaluated together with the patient history and other clinical diagnostic evaluation.

12. ANALYTICAL SPECIFICITY

5 samples (2 Negative, 1 Cut-Off and 2 Positive) were spiked with the following potentially interfering factors and then tested:

Rheumatoid Factor (44 – 220 IU/ml)

Bilirubin (4.5 mg/dl – 18 mg/dl)

Triglycerides (10 mg/dl – 150 mg/dl)

Hemoglobin (5 mg/ml – 30 mg/ml)

The presence in the serum sample of the interfering substances described above does not affect the test result.

13. CROSS-REACTIONS

54 samples, positive to Dengue, Cytomeglaovirus, Toxoplasma, Rubella ed Epstein-Barr virus were tested.

No significant cross-reactions were found except for Dengue and Epstein-Barr virus.

14. METHOD COMPARISON

In an experimentation 144 samples have been tested with Diesse kit and with a competitor kit.

Data are summarized in the following table:

		Reference		
		+	-	Total
		49	0	49
Diesse	+	49	0	49
	-	0	95	95
	Total	49	95	144

Percent Positive Agreement (~Diagnostic Sensitivity):

100% CI_{95%}: 92.7-99.9

Percent Negative Agreement: (~Diagnostic Specificity):

100% CI_{95%}: 96.1-99.9

The agreement between the two methods is excellent with a Cohen's Kappa of 1.0.

15. PRECISION AND REPEATABILITY

Sample	Within-run Precision	
	Mean (Index)	CV%
1	3.8	11.3
2	3.3	9.4
3	2.2	11.4
4	0.8	12.5

Sample	Between-run precision	
	Mean (Index)	CV%
1	3.8	14.5
2	3.7	14.6
3	2.7	11.9
4	0.8	10.0

Sample	Precision between batches		Precision between instruments	
	Mean (Index)	CV%	Mean (Index)	CV%
1	3.4	6.2	3.9	14.1
2	3.2	5.0	3.7	12.4
3	2.3	11.7	2.8	14.6
4	0.7	11.4	0.8	15.0

16. REFERENCES

1. Abrahm & Shridharan "Chikungunya Virus Infection a Resurgent Scourge" Indian J. med. Res. 126, December 2007, pp 502-504
2. M. Chhabra, V. Mittal, D. Bhattacharya, UVS Rana, S Lal "Chikungunya Fever: a re-emerging viral infection" Indian J. of Medical Microbiology, 2008- 26(1): pp 5-12

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DIESSE Diagnostica Senese S.p.A.