TEST DEVICE 10K

REF 10290

TEST DEVICE 5 K

REF 10291

TEST DEVICE 1K

REF 10292

Prodotto da/Manufactured by/Fabricado por/Fabriqu   par/
 Producent:

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1. INTENDED USE

Dedicated tube with Transponder RF.

To be used only once for the **recharging of erythrocyte sedimentation rate tests performed on a VES-MATIC CUBE instruments.**

2. INTRODUCTION AND PRINCIPLE OF THE METHOD

The insertion of Test Device (radio frequency) permits the recharging of the instrument for the amount of tests indicated in the **REF**

3. CONTENTS

Test Device 10K (cod. 10290)	1 tube containing 10.000 tests
Test Device 5K (cod. 10291)	1 tube containing 5.000 tests
Test Device 1K (cod. 10292)	1 tube containing 1.000 tests

4. WARNING

IMPORTANT:

Test Device can only be used once. After being used for the recharging, the instruments of the VES-MATIC CUBE series will not recognize the tube again.

Once used, Test Device must be disposed of in accordance with the regulations concerning electronic devices.

5. PROCEDURE

The VES-MATIC CUBE instruments are provided with a RF reader that reads the data contained in the transponder and updates the internal meter recording the number of tests that can be performed. This number corresponds to the amount of tests indicated of the Test Device code number.

For the recharging procedure refer to the Operating Manual of the used instrument.

6. LIMITATIONS OF THE PRODUCT

Test Device can only be used on the instruments of the VES-MATIC CUBE series.

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