CHORUS

TREPONEMA IgG

REF 81050

REF 81050/12

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15. BIBLIOGRAFIA

1. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
2. S.A. Lukehart et al. Characterization of monoclonal antibodies to Treponema pallidum. J. Immunology 134: 585 (1985).
3. S.A. Lukehart: Identification and characterization of Treponema pallidum antigens by monoclonal antibodies. in: Monoclonal Antibodies, Academic Press 1986, p. 1.

4. L. Lewis et al.: Evaluation of immunoglobulin M Western blot analysis in the diagnosis of congenital syphilis. *J. Clin. Microbiol.* 28: 296 (1990).
5. M. Norgard et al.: Sensitivity and specificity of monoclonal antibodies directed against antigenic determinants of *Treponema pallidum* Nichols in the diagnosis of syphilis. *J. Clin. Microbiol.* 20: 711 (1984).
6. M.J. Bailey et al.: Production of murine monoclonal antibodies to the major axial filament polypeptide of *Treponema pallidum*. *J. Gen. Microbiol.* 133: 1805 (1987)

INSTRUCTIONS FOR USE

CHORUS TREPONEMA IgG

(English)

1. INTENDED USE

QUALITATIVE DETERMINATION OF IgG-CLASS ANTIBODIES TO TREPONEMA PALLIDUM IN HUMAN SERUM, USING A DISPOSABLE DEVICE APPLIED ON THE CHORUS INSTRUMENTS

2. INTRODUCTION

The serological diagnosis of Syphilis is performed by demonstrating the presence of significant levels of specific Treponema pallidum antibodies in the serum sample.

The reference method used is the FTA-ABS technique which allows the detection of both IgG and IgM. However, its execution is laborious and the interpretation of the results is not simple; alternative methods have therefore been introduced to simplify the procedure. ELISA gives completely automated, reproducible and specific results, and for this reason is favoured by many operators.

The specific assay of IgM as opposed to IgG is of particular importance in the diagnosis of congenital Syphilis.

3. PRINCIPLE OF THE METHOD

The test is based on the ELISA principle (Enzyme linked Immunosorbent Assay). A pool of recombinant T. pallidum antigens obtained in E. coli is bound to the solid phase. The specific immunoglobulins are bound to the antigen through incubation with dilute human serum.

After washings to eliminate the proteins which have not reacted, incubation is performed with the conjugate, composed of human IgG monoclonal antibodies conjugated to horse radish peroxidase.

The unbound conjugate is eliminated, and the peroxidase substrate added.

The colour which develops is proportional to the concentration of specific antibodies present in the serum sample.

The disposable devices contain all the reagents to perform the test when applied on the Chorus instruments.

4. KIT COMPOSITION AND REAGENT PREPARATION

The kit is sufficient for 36 tests (REF 81050).

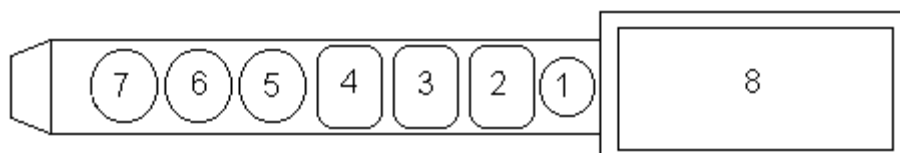
The kit is sufficient for 12 tests (REF 81050/12).

DD DEVICES

6 packages each containing 6 devices (REF 81050).

2 packages each containing 6 devices (REF 81050/12).

Use: equilibrate a package at room temperature, open the package and remove the devices required; replace the others in the bag with the silica gel, expel the air and seal by pressing the closure. Store at 2-8°C.

Description:

Position 8: Space for application of bar code label

Position 7: empty

Position 6: MICROPLATE WELL coated with a pool of T. pallidum antigens

Position 5: Uncoated MICROPLATE WELL

Position 4: TMB SUBSTRATE

Contents: Tetramethylbenzidine 0.26 mg/mL and H₂O₂ 0.01% stabilized in 0.05 mol/L citrate buffer (pH 3.8)

Position 3: SAMPLE DILUENT

Contents: Proteic solution containing phenol 0.05%, Bronidox 0.02% and an indicator to reveal the presence of the serum.

Position 2: CONJUGATE

Contents: anti-human IgG monoclonal antibodies labelled with horseradish peroxidase, in phosphate buffer containing phenol 0.05% and Bronidox 0.02%

Position 1: EMPTY WELL in which the operator must place the undiluted serum

CALIBRATOR CALIBRATOR 1 x 0.425 mL

Contents: Diluted human serum, at known antibody concentration, containing Proclin and Gentamycin, **liquid ready for use.**

CONTROL + POSITIVE CONTROL 1 x 0.425 mL

Contents: Diluted human serum, at known antibody concentration, containing Proclin and Gentamycin, **liquid ready for use.**

MATERIALS REQUIRED BUT NOT PROVIDED

- WASHING BUFFER **REF** 83606
- CLEANING SOLUTION 2000 **REF** 83609
- SANITIZING SOLUTION **REF** 83604 – 83608
- CHORUS NEGATIVE CONTROL/SAMPLE DILUENT **REF** 83607
- Distilled or deionised water
- Normal laboratory glassware: cylinders, test-tubes etc.
- Micropipettes for the accurate collection of 50-200 µl solution
- Disposable gloves
- Sodium Hypochlorite solution (5%)
- Containers for collection of potentially infectious materials

5. STORAGE AND STABILITY OF REAGENTS

Reagents must be stored at 2/8°C. In the case of storage at an incorrect temperature the calibration must be repeated and the run validated using the control serum (see point 9, Test validation) .

The expiry date is printed on each component and on the kit label.

Reagents have a limited stability after opening and/or preparation:

DEVICES	8 weeks at 2/8°C
CALIBRATOR	8 weeks at 2/8°C
POSITIVE CONTROL	8 weeks at 2/8°C

6. PRECAUTIONS

ONLY FOR DIAGNOSTIC USE IN VITRO.

Attention:

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HbsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling material of human origin.

Waste disposal: serum samples, calibrators and strips once used must be treated as infectious residuals and eliminated according to law.

Health and Safety Information

1. Do not pipette by mouth. Wear disposable gloves and eye protection while handling specimens. Wash hands thoroughly after placing the devices in the CHORUS instrument.
2. The following reagents contain low concentrations of harmful or irritant substances:
 - a) The conjugate contains phenol
 - b) The substrate is acid
 If any of the reagents come into contact with the skin or eyes, wash the area extensively with water.
3. Neutralized acids and other liquid waste should be decontaminated by adding a sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1%. A 30 minute exposure to 1% sodium hypochlorite may be necessary to ensure effective decontamination.

4. Spillage of potentially infectious materials should be removed immediately with adsorbent paper tissue and the contaminated area swabbed with, for example, 1% sodium hypochlorite before work is continued. Sodium hypochlorite should not be used on acid-containing spills unless the spill area is first wiped dry. Materials used to clean spills, including gloves, should be disposed of as potentially biohazardous waste. Do not autoclave materials containing sodium hypochlorite.

Analytical precautions

Bring the devices to be used to room temperature (18-30°C) before use; use within 60 min.

1. **Discard devices which show the substrate (well 4) blue colored.**
2. Adding the sample into the well verify that it is perfectly distributed on the bottom.
3. Check for the presence of the reagents in the device and that the device is not damaged; do not use devices which are lacking a reagent.
4. The devices are for use with the Chorus instrument; the instructions for use must be carefully followed and the instrument operating manual must be consulted.
5. Check that the Chorus instrument is set up correctly (see Chorus Operating Manual).
6. Do not alter the bar code placed on the handle of the device in order to allow correct reading by the instrument
7. Defective barcodes can be inserted manually in the instrument.
8. Do not expose the devices to strong light or to hypochlorite vapours during storage and use.
9. The use of strongly hemolyzed samples, or samples presenting microbial contamination may all constitute sources of error.
10. Before inserting the devices in the instrument, check that the reaction well does not contain foreign bodies.
11. Pipette the test serum (50 µl) in well 1 of the device (see figure).
12. Do not use the device after the expiry date

7. TYPE AND STORAGE OF SAMPLE

The sample is composed of serum collected in the normal manner from the vein and handled with all precautions dictated by good laboratory practice. The fresh serum may be stored for 4 days at 2/8°C, or frozen for longer periods at -20°C, and can be thawed a maximum of 3 times. Do not keep the samples in auto-defrosting freezers. Defrosted samples must be shaken carefully before use. The quality of the sample can be seriously affected by microbial contamination which leads to erroneous results.

Strongly lipemic, icteric, hemolyzed or contaminated samples should be avoided.

The test cannot be applied to plasma.

8. PROCEDURE

1. Open the package (on the side containing the pressure-closure), remove the number of devices required and seal the rest in the bag after expelling the air.
2. Check the state of the device according to the indications reported in the Analytical Warnings points 1 and 8.
3. Dispense 50 µl of undiluted test serum in well no. 1 of each device; at each change of batch, use a device for the calibrator.
4. Place the devices in the instrument. Perform the calibration (if necessary) and the test as reported in the Chorus Operating Manual.

9. TEST VALIDATION

Use the control serum to check the validity of the results obtained. It should be used as reported in the operating manual. If the instrument signals that the control serum has a value outside the acceptable range, the calibration must be repeated. The previous results will be automatically corrected.

If the result of the control serum continues to be outside the acceptable range, contact the Scientific Support.

Tel: 0039 0577 319554
 Fax: 0039 0577 366605
 email: scientificsupport@diesse.it

10. INTERPRETATION OF THE RESULTS

The Chorus instrument expresses the result as an INDEX (ratio between the OD value of the test sample and that of the Cut-Off) which can be used as a quantitative measure, as it is proportional to the amount of specific IgG present.

The test serum can be interpreted as follows:

NEGATIVE: when the result is < 0.8

DOUBTFUL/EQUIVOCAL: when the result is between 0.8 and 1.2.

POSITIVE: when the ratio is > 1.2

In the case of a doubtful/equivocal result, repeat the test. If it remains doubtful/equivocal, take a new sample.

11. LIMITATIONS OF THE PROCEDURE

A serum sample obtained during the acute phase of infection, when only IgM antibodies are present, may be negative by this procedure.

The Syphilis IgM level should be determined using the Syphilis IgM kit (REF 81051). Alternatively, a second serum sample obtained 8-14 days later, should be tested in parallel to determine an increase in the IgG antibody level.

The test result should be used in conjunction with information available from the evaluation of the case history or other diagnostic procedures.

There is a possibility of cross-reactions in samples containing anti-E. coli antibodies.

12. ANALYTICAL SPECIFICITY

25 serum samples containing antibodies anti-Borrelia burgdorferi, 12 containing anti-nuclear antibodies (ANA), 12 containing anti-EBV antibodies were tested. The same samples were tested with two other commercial methods. Only one sample (EBV) gave a positive reaction in one of the other 2 commercial methods.

13. DIAGNOSTIC SENSITIVITY AND SPECIFICITY

209 samples from a serum collection were tested. Another immunoenzymatic method was used for comparative purposes; discordant samples were tested with a Syphilis Screen Recombinant kit (ref. 91100 Diesse).

The results are summarized in the following table.

		REFERENCE	
		+	-
+		43	1
DIESSE -		1	164

The kit shows a sensitivity of 98% (CI 95%: 88–100) and a specificity of 99% (CI 95%: 97–100).

14. PRECISION

Within-run precision

	No. of replicates	Mean Index	St. Dev.	CV%
Lot no. 018	6	2.5	0.28	11
Lot no. 021	9	2.9	0.35	12
Lot no. 022	9	3.3	0.24	7

Between-run precision and Precision between batches

Sample	Mean Index			Mean	St. Dev.	CV%
	Lot 018	Lot 021	Lot 022			
TPG 1	0.2	0.2	0.2	0.2	0.00	-
TPG 2	1.4	1.3	1.8	1.5	0.26	17
TPG 3	4.6	3.9	4.4	4.3	0.36	8

15. REFERENCES

1. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
2. S.A. Lukehart et al. Characterization of monoclonal antibodies to Treponema pallidum. J. Immunology 134: 585 (1985).
3. S.A. Lukehart: Identification and characterization of Treponema pallidum antigens by monoclonal antibodies. in: Monoclonal Antibodies, Academic Press 1986, p. 1.
4. L. Lewis et al.: Evaluation of immunoglobulin M Western blot analysis in the diagnosis of congenital syphilis. J. Clin. Microbiol. 28: 296 (1990).
5. M. Norgard et al.: Sensitivity and specificity of monoclonal antibodies directed against antigenic determinants of Treponema pallidum Nichols in the diagnosis of syphilis. J. Clin. Microbiol. 20: 711 (1984).
6. M.J. Bailey et al.: Production of murine monoclonal antibodies to the major axial filament polypeptide of Treponema pallidum. J. Gen. Microbiol. 133: 1805 (1987).

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15. POUŽITÁ LITERATURA

- G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
- S.A. Lukehart et al. Characterization of monoclonal antibodies to *Treponema pallidum*. J. Immunology 134: 585 (1985).
- S.A. Lukehart: Identification and characterization of *Treponema pallidum* antigens by monoclonal antibodies. in: Monoclonal Antibodies, Academic Press 1986, p. 1.
- L. Lewis et al.: Evaluation of immunoglobulin M Western blot analysis in the diagnosis of congenital syphilis. J. Clin. Microbiol. 28: 296 (1990).
- M. Norgard et al.: Sensitivity and specificity of monoclonal antibodies directed against antigenic determinants of *Treponema pallidum* Nichols in the diagnosis of syphilis. J. Clin. Microbiol. 20: 711 (1984).
- M.J. Bailey et al.: Production of murine monoclonal antibodies to the major axial filament polypeptide of *Treponema pallidum*. J. Gen. Microbiol. 133: 1805 (1987).

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15. LITERATUR

1. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
2. S.A. Lukehart et al. Characterization of monoclonal antibodies to *Treponema pallidum*. J. Immunology 134: 585 (1985).
3. S.A. Lukehart: Identification and characterization of *Treponema pallidum* antigens by monoclonal antibodies. in: Monoclonal Antibodies, Academic Press 1986, p. 1.
4. L. Lewis et al.: Evaluation of immunoglobulin M Western blot analysis in the diagnosis of congenital syphilis. J. Clin. Microbiol. 28: 296 (1990).
5. M. Norgard et al.: Sensitivity and specificity of monoclonal antibodies directed against antigenic determinants of *Treponema pallidum* Nichols in the diagnosis of syphilis. J. Clin. Microbiol. 20: 711 (1984).
6. M.J. Bailey et al.: Production of murine monoclonal antibodies to the major axial filament polypeptide of *Treponema pallidum*. J. Gen. Microbiol. 133: 1805 (1987).

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15. BIBLIOGRAFIA

1. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
2. S.A. Lukehart et al. Characterization of monoclonal antibodies to *Treponema Pallidum*. J. Immunology 134: 585 (1985).
3. S.A. Lukehart: Identification and characterization of *Treponema Pallidum* antigens by monoclonal antibodies. in: Monoclonal Antibodies, Academic Press 1986, p. 1.
4. L. Lewis et al.: Evaluation of immunoglobulin M Western blot analysis in the diagnosis of congenital syphilis. J. Clin. Microbiol. 28: 296 (1990).
5. M. Norgard et al.: Sensitivity and specificity of monoclonal antibodies directed against antigenic determinants of *Treponema Pallidum* Nichols in the diagnosis of syphilis. J. Clin. Microbiol. 20: 711 (1984).
6. M.J. Bailey et al.: Production of murine monoclonal antibodies to the major axial filament polypeptide of *Treponema pallidum*. J. Gen. Microbiol. 133: 1805 (1987).

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15. BIBLIOGRAPHIE

1. G.B. Wisdom: Enzyme-Immunoassay. Clin. Chem. 22: 1243 (1976).
2. S.A. Lukehart et al. Characterization of monoclonal antibodies to Treponema Pallidum. J. Immunology 134: 585 (1985).
3. S.A. Lukehart: Identification and characterization of Treponema Pallidum antigens by monoclonal antibodies. in: Monoclonal Antibodies, Academic Press 1986, p. 1.
4. L. Lewis et al.: Evaluation of immunoglobulin M Western blot analysis in the diagnosis of congenital syphilis. J. Clin. Microbiol. 28: 296 (1990).

5. M. Norgard et al.: Sensitivity and specificity of monoclonal antibodies directed against antigenic determinants of *Treponema Pallidum* Nichols in the diagnosis of syphilis. *J. Clin. Microbiol.* 20: 711 (1984).
6. M.J. Bailey et al.: Production of murine monoclonal antibodies to the major axial filament polypeptide of *Treponema pallidum*. *J. Gen. Microbiol.* 133: 1805 (1987).

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