DIESSE Diagnostica Senese S.p.A.

Strada dei Laghi 39, 53035 Monteriggioni (SI)

Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

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**Accessory reagent for use in the
autoimmunity kits in CHORUS
instruments**

For *in vitro* diagnostic use only

REF 86004 (EN)

1. INTENDED PURPOSE

FOR USE IN THE AUTOIMMUNITY TESTS IN THE CHORUS ANALYZERS AS WASH SOLUTION FOR THE REACTION WELLS.

IT MUST ONLY BE USED BY PROFESSIONAL LABORATORY PERSONNEL.

2. COMPOSITION

WASH BUF 50X WASH BUFFER 50X 12x20 mL.

INTERCHANGEABLE BETWEEN BATCHES

Contents: Wash solution concentrated 50 times.

Preparation: Dilute the required volume 1:50 with distilled water to obtain the wash buffer ready for use. If crystals are present, dissolve them at 37°C before dilution.

3. USE

Follow the instructions for use of the kit in which the reagent is to be adopted. In CHORUS instruments, pour the ready-to-use reagent into the tank with the blue sleeve.

4. STABILITY AFTER OPENING

The solution is stable up to the expiry date if maintained at 2-30 °C.

The ready-to-use solution is stable up to 2 weeks at 2/8°C, 5 days at 15/30 °C.

5. WARNING

Regarding the safety characteristics, consult the Safety Data Sheet (www.diesse.it).

6. REFERENCES

- Engvall E, Perlman P (1971). "Enzyme-linked immunosorbent assay (ELISA). Quantitative assay of immunoglobulin G". *Immunochemistry* **8** (9): 871-4

7. INCIDENT REPORTING

If a serious incident has occurred in relation to this device in the European Union market territory, please report it to the manufacturer and the competent authority of its Member State without delay.

	Capitolo/Section/Capítulo/ Kapitel/Κεφάλαιο/ Chapitre/Kapitola/Seitiune
Modifiche introdotte nella revisione corrente/Changes introduced in the current revision Cambios introducidos en la revisión actual/Τροποποιήσεις που εισήχθησαν στην τρέχουσα διόρθωση Eingeführte Änderungen in der vorliegenden Fassung/Modifications apportées à la révision courante Změny provedené v aktuální verzi/Modificari introduse in versiunea curenta Alterações introduzidas na revisão atual	1 – 5 – 7

**DIESSE Diagnostica Senese SpA**

Strada dei Laghi 39, 53035 Monteriggioni (SI)
Tel. ++39/0577/ 587111 - Fax ++39/0577/318690

CHORUS WASHING BUFFER AUTOIMMUNITY

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