DIESSE Diagnostica Senese S.p.A.

Strada dei Laghi 39, 53035 Monteriggioni (SI)
Tel. + + 39/0577/319560/61 - Fax + + 39/0577/318763

ENZY-WELL/CHORUS WASHING BUFFER

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Accessory reagent for use in Enzy-Well immunoenzymatic kits and CHORUS instruments

For *in vitro* diagnostic use only

REF 83606 (EN)

1. INTENDED PURPOSE

TO BE USED IN THE ENZY-WELL IMMUNOENZYMATIC TEST KITS AND CHORUS INSTRUMENTS AS WASH SOLUTION FOR THE REACTION WELLS.

IT MUST ONLY BE USED BY PROFESSIONAL LABORATORY PERSONNEL.

2. COMPOSITION

WASH BUF 10x WASH BUFFER 10X 4 x 100 mL. **INTERCHANGEABLE BETWEEN BATCHES**

Contents: Buffered saline solution (PBS) concentrated 10 times, containing Brij 0.5% .

Preparation: Dilute the required volume 1:10 with distilled water to obtain the wash buffer ready for use. If crystals are present, dissolve them at 37°C before dilution.

3. USE

Follow the instructions for use of the kit in which the reagent is to be adopted. In CHORUS instruments, pour the ready-to-use reagent into the tank with the blue sleeve.

4. STABILITY AFTER OPENING

The solution is stable up to the expiry date if maintained at 2-30 °C.

The ready-to-use solution is stable up to 2 weeks at 2/8°C, 5 days at 15/30 °C.

5. WARNING

The reagent contains low concentrations of harmful or irritating substances: the Wash Buffer contains detergents.

6. REFERENCES

- Engvall E, Perlman P (1971). "Enzyme-linked immunosorbent assay (ELISA). Quantitative assay of immunoglobulin G". *Immunochemistry* **8** (9): 871-4

7. INCIDENT REPORTING

If a serious incident has occurred in relation to this device in the European Union market territory, please report it to the manufacturer and the competent authority of its Member State without delay.

		Capitolo/Section/ Capítulo/Κεφάλαιο/ Chapitre/Kapitola/Sectiune/ Poglavlje / Глава / Peatükk / Skyrius / Rozdział
Modifiche introdotte nella revisione corrente/Changes introduced in the current revision Cambios introducidos en la revisión actual/Τροποποιήσεις που εισήχθησαν στην τρέχουσα διάρθρωση Modifications apportées à la révision courante/Změny provedené v aktuální verzi/Modificări introduse în versiunea curentă/Alterações introduzidas na revisão atual / Promjene uvedene u aktualnoj reviziji / Промени, въведени в настоящата ревизия / Käesoleva läbivaatusega sisse viidud muudatused / Dabartinėje redakcijoje atlikti pakeitimai/ Zmiany wprowadzone w bieżącej aktualizacji		1-5-7

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