CHORUS CLEANING SOLUTION 2000

REF 83609

DIESSE

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INSTRUCTIONS FOR USE

CHORUS CLEANING SOLUTION 2000

**Accessory reagent for use in CHORUS
instruments**

For *In vitro* Diagnostic Use Only

1. INTENDED PURPOSE

To be used in CHORUS instruments as a wash solution for the tubes and as resuspending solution for the lyophilized products of the CFT line.

It must only be used by professional laboratory personnel.

CLEAN SOL 2000x CLEANING SOLUTION
2000X 4 x 20 mL **INTERCHANGEABLE
BETWEEN BATCHES**

Contents: Saline solution containing NaCl 12%.

Preparation: Dilute the required volume 1:2000 with distilled water to obtain the cleaning solution ready for use.

2. USE

In CHORUS instruments, pour the ready-to-use solution into the 2-litre tank to be connected to the inlet with the white circle.

3. STABILITY AFTER OPENING

The solution is stable up to the expiry date if maintained at 2-30°C.

The ready-to-use solution is stable up to 5 days at room temperature of 15-30 °C.

4. WARNING

When collecting pay attention to not contaminate the product.

Regarding the safety characteristics, consult the Safety Data Sheet (www.diesse.it).

5. INCIDENT REPORTING

If a serious incident has occurred in relation to this device in the European Union market territory, please report it to the manufacturer and the competent authority of its Member State without delay.

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