DIESSE Diagnostica Senese SpA
Strada dei Laghi 39, 53035 Monteriggioni (SI)
Tel. + + 39/0577/319560/61 - Fax + + 39/0577/318763

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(EN)

1. INTENDED PURPOSE

TO BE USED IN IMMUNOENZYMATIC TESTS IN CHORUS INSTRUMENTS AS NEGATIVE CONTROL OR DILUENT OF SAMPLES THAT GAVE OUT-OF-RANGE RESULTS.

IT MUST ONLY BE USED BY PROFESSIONAL LABORATORY PERSONNEL.

For *in vitro* diagnostic use only

2. INTRODUCTION

The use of quality control materials is recommended as an objective evaluation of the precision of the method and is an integral part of good laboratory practice. This product is used as a test sample, or as diluent for over-range samples.

3. REAGENT

CONTROL - NEGATIVE CONTROL / SAMPLE DILUENT
1 x 5 ml. Ready for use

Contents: Stabilized proteic solution.

Stability: The unopened product is stable up to the expiry date if maintained at 2/8 °C. After opening the product is stable for 2 months at 2/8°C and for 1 week at 15/30°C; store in the dark.

Procedure: Bring to room temperature before use. Use as reported in the Chorus Instruction manual when using as Negative Control. When used as sample diluent, follow standard good laboratory practice.

4. WARNINGS

- The product must not be used after the expiry date.
- In the case of evident microbial contamination or marked turbidity of the product, eliminate the vial.
- The product is not for use as a standard.
- In an internal study, the Negative Control gave an index close to the doubtful limit in the test REF 81051 Treponema IgM. Its use as Negative Control is not recommended in this test.

5. REFERENCES

- Engvall E, Perlman P (1971). "Enzyme-linked immunosorbent assay (ELISA). Quantitative assay of immunoglobulin G". *Immunochemistry* **8** (9): 871-4

6. INCIDENT REPORTING

If a serious incident has occurred in relation to this device in the European Union market territory, please report it to the manufacturer and the competent authority of its Member State without delay.

	Capitolo/Section/ Capitulo/Κεφάλαιο/ Chapitre/Kapitola/Sección/ Poglavlje / Глава / Peatükk / Skyrius / Rozdział
Modifiche introdotte nella revisione corrente/Changes introduced in the current revision Cambios introducidos en la revisión actual/Τροποποιήσεις που εισήχθησαν στην τρέχουσα διόρθωση Modifications apportées à la révision courante/Změny provedené v aktuální verzi/Modificari introduse in versiunea curenta/Alterações introduzidas na revisão atual / Promjene uvedene u aktualnoj reviziji / Промени, въведени в настоящата ревизия / Käesoleva läbivaatusega sisse viidud muudatused / Dabartinėje redakcijoje atlikti pakeitimai/ Zmiany wprowadzone w bieżącej aktualizacji	1 - 6

DIESSE Diagnostica Senese SpA
Strada dei Laghi 39, 53035 Monteriggioni (SI)
Tel. + + 39/0577/319560/61 - Fax + + 39/0577/318763

CHORUS NEGATIVE CONTROL/SAMPLE DILUENT

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