**DIESSE Diagnostica Senese SpA**  
Via delle Rose 10, 53035 Monteriggioni SI  
Tel. ++39/0577/587111 - Fax ++39/0577/318690

### CHORUS CONTROL SERA

Prodotti compresi nell'All. IIB, Dir. 98/79 CE Product included in the Annex IIB, Dir. 98/79 CE	REF
CHORUS CYTOMEGALOVIRUS IgG CONTROL SERUM	81500
CHORUS CYTOMEGALOVIRUS IgG AVIDITY CONTROL SERUM	81501
CHORUS CYTOMEGALOVIRUS IgM CONTROL SERUM	81502
CHORUS RUBELLA IgG CONTROL SERUM	81523
CHORUS RUBELLA IgG AVIDITY CONTROL SERUM	81524
CHORUS RUBELLA IgM CONTROL SERUM	81525
CHORUS TOXOPLASMA IgG CONTROL SERUM	81531
CHORUS TOXOPLASMA IgG AVIDITY CONTROL SERUM	81532
CHORUS TOXOPLASMA IgM CONTROL SERUM	81533
CHORUS TOXOPLASMA IgA CONTROL SERUM	81534
CHORUS CHLAMYDIA TRACHOMATIS IgG CONTROL SERUM	81542
CHORUS CHLAMYDIA TRACHOMATIS IgA CONTROL SERUM	81543
Notified Body Code Number 0123	
 0123	
Prodotto Product	REF
CHORUS EPSTEIN-BARR VCA IgG CONTROL SERUM	81503
CHORUS EPSTEIN-BARR VCA IgM CONTROL SERUM	81504
CHORUS EPSTEIN-BARR EBNA IgG CONTROL SERUM	81505
CHORUS EPSTEIN-BARR EARLY ANT. IgG CONTROL SERUM	81506
CHORUS EPSTEIN-BARR EARLY ANT. IgM CONTROL SERUM	81507

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##### 6. BIBLIOGRAFIA

1. C24-A3 (ISBN 1-56238-613-1), Clinical and Laboratory Standard Institute (CLSI), Vol. 26 No.25, (2006)

CHORUS HELICOBACTER PYLORI IgG CONTROL SERUM	81508
CHORUS HELICOBACTER PYLORI IgA CONTROL SERUM	81509
CHORUS HSV 1 SCREEN CONTROL SERUM	81510
CHORUS HERPES SIMPLEX 1+2 IgG CONTROL SERUM	81512
CHORUS HERPES SIMPLEX 1+2 IgM CONTROL SERUM	81513
CHORUS HERPES SIMPLEX 1 IgG RECOMBINANT CONTROL SERUM	81537
CHORUS HERPES SIMPLEX 2 IgG RECOMBINANT CONTROL SERUM	81539
CHORUS MEASLES IgG CONTROL SERUM	81517
CHORUS MUMPS IgG CONTROL SERUM	81519
CHORUS MUMPS IgM CONTROL SERUM	81520
CHORUS MYCOPLASMA PNEUMONIAE IgG CONTROL SERUM	81521
CHORUS MYCOPLASMA PNEUMONIAE IgM CONTROL SERUM	81522
CHORUS SYPHILIS SCREEN RECOMBINANT CONTROL SERUM	81526
CHORUS TREPONEMA IgG CONTROL SERUM	81527
CHORUS TREPONEMA IgM CONTROL SERUM	81528
CHORUS VARICELLA IgG CONTROL SERUM	81535
CHORUS VARICELLA IgM CONTROL SERUM	81536
CHORUS TETANUS IgG CONTROL SERUM	81540
CHORUS DIPHTHERIA IgG CONTROL SERUM	81541
CHORUS PARVOVIRUS B19 IgG CONTROL SERUM	81544
CHORUS PARVOVIRUS B19 IgM CONTROL SERUM	81545

#### Instructions for use of the Control Sera

(EN)

##### 1. INTENDED USE

Control material to be used exclusively in the control of precision of the Chorus/Chorus TRIO instruments.

##### 2. COMPOSITION

**CONTROL + POSITIVE CONTROL** 2 x 0.4 mL

Contents: Reactive human serum for the parameters reported above.

Preparation: ready for use. To be used as any other sample.

##### 3. STORAGE AND STABILITY OF REAGENT

Control sera must be stored at 2-8°C.

The expiry date is printed on the label.

Once opened, if properly stored at 2-8°C, the product is stable up to the expire date.

##### 4. PRECAUTIONS

###### ONLY FOR IN VITRO DIAGNOSTIC USE

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HbsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling material of human origin.

Waste disposal: serum samples once used must be treated as infectious residuals and eliminated according to law.

##### 5. PROCEDURE

To be used use as any other sample being tested.

##### 6. REFERENCES

1. C24-A3 (ISBN 1-56238-613-1), Clinical and Laboratory Standard Institute (CLSI), Vol. 26 No.25, (2006)

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**DIESSE Diagnostica Senese S.p.A.**